

## Backup Instructions For Zepto<sup>®</sup> Release

Zepto<sup>®</sup> is designed to be released off the capsule by pushing the roller all the way forward (away from the red clamp). This action re-introduces ~0.2cc of OVD and BSS into the suction cup, floating Zepto<sup>®</sup> off the capsule.

Release may fail if:

The suction line was not primed or incompletely primed at the beginning of the procedure with BSS,

The red clamp was inadvertently closed before pushing the roller forward,

Or there is a kink in line.

If release did not occur after pushing the roller forward, follow these instructions:

Instruction	Comment
Check that the red clamp is open and there are no kinks in the suction line.	Ensures fluid flow in suction line.
<b>Step 1</b> Detach the isolator from the roller mechanism.	Allows syringe attachment in step 3.
<b>Step 2</b> Reset roller back towards the red clamp.	Opens up suction line.
<b>Step 3</b> <ul style="list-style-type: none"> <li>• Attach sterile syringe filled with sterile BSS to roller assembly.</li> <li>• Slowly inject ~0.2cc of BSS as the surgeon watches Zepto<sup>®</sup> tip floats off the capsule.</li> </ul>	Re-introduces OVD and BSS to float Zepto <sup>®</sup> off the capsule.