

Guidelines for OVD Use During Zepto® Capsulotomy

Do not use

OVD	Viscosity (mPas)
Viscoadaptives	
Healon 5	7 M
MicroVisc (iVisc) Plus	3.3 M
Super viscous viscoelastic	
Healon GV	3 M
Viscous-cohesive viscoelastic	
MicroVisc (iVisc) Phaco	1 M
MicroVisc (iVisc)	1 M
Allervisc	500 K

OVD viscosity cut-off is
300,000 mPas (or 300,000 cps)

Use

Cohesive	
Healon	300 K
Provisc	280 K
Biolon	215 K
Allervisc	200 K
Amvisc	100 K
Amvisc Plus	60 K
Viscous dispersive	
DisCoVisc	100 K
Dispersive	
Healon Endocoat	50 K
Viscoat	50 K
NuVisc	45 K
Cellugel	40K
Vitrax	25 K
Dispersive very low viscosity	
i-Cell	6 K
Ocuvis	4.3 K
Ocucoat	4 K
Hymecel	4K
Adatocel	4 K
Visilon	4 K

- Any OVD not listed here with a viscosity of less than or equal to 300K mPas (or 300K cps) may be used during Zepto® capsulotomy
- Any OVD may be used after Zepto® capsulotomy