



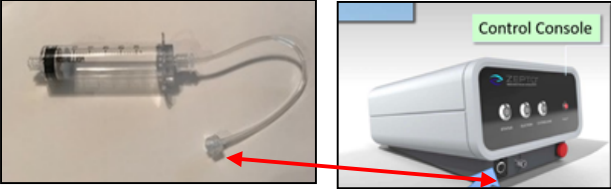

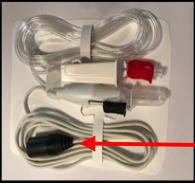

Zepto® Set-Up Flowchart

	Scrub Assistant (Sterile)	Circulating Assistant (Non-Sterile)
Step 1	<ul style="list-style-type: none"> Set up sterile field per normal protocol. 	<ul style="list-style-type: none"> Attach power cord and turn on console. A successful self-check will result in a FLASHING GREEN light.
Step 2		<ul style="list-style-type: none"> Place sterile handpiece pack onto sterile field. Open and attach non-sterile fluid isolator to console. 
Step 3	<ul style="list-style-type: none"> Open sterile handpiece pack. Pass the electrical line's black connector to non-sterile assistant. 	<ul style="list-style-type: none"> Plug black connector of electrical line into console. (FLASHING GREEN light will change to STEADY GREEN) 
Step 4	<p>METHOD 1</p> <ul style="list-style-type: none"> Connect handpiece suction line to irrigation line from phaco machine. Prime handpiece suction line with BSS from phaco irrigation line until no air remains in suction line. Turn off irrigation and close red clamp on handpiece suction line. Pass clamped line off to non-sterile assistant  <p style="text-align: center;"><i>OR</i></p> <p>METHOD 2</p> <ul style="list-style-type: none"> Fill a 10CC syringe with sterile BSS. Connect syringe to handpiece suction line. Prime handpiece suction line with BSS from syringe. Continue until BSS comes out through the handpiece's protective cover and no air remains in suction line. Remove syringe and close red clamp on handpiece suction line. Pass clamped line off to non-sterile assistant 	
Step 5	<p style="text-align: center;">Zepto® is now ready for use</p>	  <ul style="list-style-type: none"> Attach clamped handpiece suction line to fluid isolator and OPEN RED CLAMP (critical) <p style="text-align: center;">Zepto® is now ready for use</p>